

Is objective ADHD assessment achievable with adolescents and adults via Telehealth?

An international audit and comparison of technical feasibility and test-taker usability of QbCheck in the home and clinic settings Presented by Lisa Cordeiro, Mikkel Hansen, Charlotte Cooper and Mallory Garcia



8th World Congress on ADHD | 6-9 May 2021

I hereby declare that I have had business or personal interests in the following industrial enterprises since 1 Apr 2020: Qbtech AB // Head of Global Clinical Research

Background and Methods



An international audit was conducted March 2020 to March 2021 to compare test-taker usability and technical feasibility of QbCheck in-clinic and in-home among adolescents and adults in Europe and the U.S.

- During this audit trained test administrators gave the test in-clinic or patients completed the test via Telehealth at home via their personal laptop.
- Test-takers were asked 10 questions about their test experience.
- Tests of proportions¹ were used to examine potential significant differences between in-clinic and in-home test issue monitoring.

QbCheck objectively measures the three core symptoms of ADHD (hyperactivity, inattention and impulsivity)². The online platform includes a DSM-V based rating scale of ADHD behaviors.

Recent reports have found:

- High equivalency across all parameters measured in-clinic and in-home³.
- Clinical validity for the prediction of ADHD diagnosis with in-home testing (83.1% sensitivity; 80.6% specificity)⁴.

^[1] R Stats Package Documentation: Test of equal or Given Proportions. Retrieved April 28, 2021, from https://stat.ethz.ch/R-manual/R-devel/library/stats/html/prop.test.html; [2] FDA cleared, K143468, K133382, K122149; [3] Ulberstad F, Boström H, Chavanon M-L, et al. Objective measurement of attention deficit hyperactivity disorder symptoms outside the clinic using the QbCheck: Reliability and validity. Int J Methods Psychiatr Res. 2020;e1822. https://doi.org/10.1002/mpr.1822; [4] Ulberstad, F., Bostrom,H., Christiansen, H., et al. Clinical validity of the QbCheck,a home based online test of objective markers associated with ADHD. Poster presented at 7th ADHD World Congress.

⁸th World Congress on ADHD | 6-9 May 2021

Results: Technical Feasibility



Of 9,213 in-home tests and 3,444 in-clinic tests, the majority of both in-home (89.3%) and in-clinic (83.7%) tests were successful.

'After Test' issues were very low and did not significantly differ between test settings (p=.058).

In-clinic 'During Test' issues were significantly higher compared to

in-home testing (p<.0001). All 'During Test' issues occurred more frequently in-clinic.

A greater percentage of in-home tests were cancelled compared to in-clinic (p<.0001).

Percentage of Issues In-Clinic and In-Home	In-clinic	In-home
Issue Identified After Test**	1.54%	1.39%
Camera too slow to follow movement	0.07%	0.29%
Camera unable to track face*	0.84%	0.56%
Stimuli presentation delayed	0.07%	0.15%
Issue Identified During Test**	13.53%	6.84%
Responses recorded without face detection**	11.80%	5.95%
No responses recorded and no face detection	0.14%	0.09%
No responses recorded**	1.49%	080%
No camera feed	0.41%	0.38%
Cancelled Test (aborted by test-taker)**	1.27%	2.49%

*p<.05; **p<.0001

Results: Test-taker Usability

Qbtech ≽

The majority of test-takers reported that they were **able** to correctly establish each of the test environment conditions in the home.

The majority of test-takers reported **in-home** testing as attainable.

94.5%

found the test **instructions** easy to understand and follow

rated the test easy or very easy 73.8% to set-up and follow through



No distractions **93%** (TV, phone, etc.)

Appropriate lighting 88%

Correct seating 74.8%

Steady table 94.1%

Seated corrected distance from laptop **92.4%**



Is objective ADHD assessment achievable Qk with adolescents & adults via Telehealth?



Yes.

The results of this year-long international audit of more than 9,000 in-home tests found that:

The majority of in-home **tests were successfully completed and technically sound.**

The majority of test-takers **rated the in-home testing process as 'easy'** and completed within standardized test conditions. These results, combined with previous findings of the equivalency¹, validity² and role of QbTest Telehealth in patient access to ADHD care³, support the utilization of QbCheck for in-home objective ADHD assessment.

[1] Ulberstad F, Boström H, Chavanon M-L, et al. Objective measurement of attention deficit hyperactivity disorder symptoms outside the clinic using the QbCheck: Reliability and validity. Int J Methods Psychiatr Res. 2020;e1822. <u>https://doi.org/10.1002/mpr.1822;</u> [2] Ulberstad, F., Bostrom,H., Christiansen, H., et al. Clinical validity of the QbCheck, a home-based online test of objective markers associated with ADHD. Poster presented at 7th ADHD World Congress.; [3] Doolin, J. and Cordeiro, L. Impact of the COVID-19 Pandemic on Objective ADHD Assessment and Telehealth Adoption: An Audit of U.S.-based Clinics and Global Investigation of Usability and Feasibility Among Pediatric and Adult Clinics Adopting QbTest Telehealth. Presented at the APSARD Annual Conference, January 15, 2021. Virtual.

8th World Congress on ADHD | 6-9 May 2021